# VS-Sense

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REFERENCES:  
1. Owen MK, Clenney TL. Management of vaginitis. Am Fam Physician. 2004 Dec 17(11):2125-32.  
2. Leitch H, Bodner-Adler B, Brunbauer M, Kaider A, Egarter C, Husslein P. Bacterial vaginosis as a risk factor for preterm delivery: a meta-analysis. Am J Obstet Gynecol. 2003 Jul;189(1):139-47.  
3. Yen S, Shafer MA, Moncada J, Campbell CJ, Flinn SD, Boyer CB. Bacterial vaginosis in sexually experienced and non-sexually experienced young women entering the military. Obstet Gynecol. 2003 Nov;102(5 Pt 1):927-33.  
4. Pavletic AJ, Hawes SE, Geske JA, Bringe K, Polack SH. Experience with routine vaginal pH testing in a family practice setting. Infect Dis Obstet Gynecol. 2004 Jun;12(2):63-8.  
5. Thinkhamrop J, Lumbiganon P, Thongkrajai P, Chongsomchai C, Pakarasang M. Vaginal fluid pH as a screening test for vaginitis. Int J Gynaecol Obstet. 1999 Aug;66(2):143-8.  
6. Egan ME, Lipsky MS. Diagnosis of vaginitis. Am Fam Physician. 2000 Sep 16(25):1095-104.  
7. CDC - Sexually Transmitted Diseases Treatment Guidelines, 2010 Recommendations and Reports December 17, 2010 / Vol. 59 / No. RR-12

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EU Authorized Representative: Obelis s.a, Boulevard Général Wabis 53, 1030  
Brussels, Belgium, www.obelis.net

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# VS-SENSE

## Vaginal Discharge Self-Test

Carefully read the entire insert before using this product

Some menopausal women may have an elevated vaginal pH, and may there for receive a green/blue result with the swab even though they are not suffering from bacterial vaginosis or trichomoniasis. As such, you should not use the test if you are menopausal.

The VS-SENSE is an in vitro self test, easy-to-use home test kit for external use only, to be used by women who sense their vaginal discharge is abnormal, to give an initial indication regarding the causes of their abnormal vaginal discharge.

A vaginal discharge is abnormal if it is excessive, has an unusual color, or has an abnormal odor, compared to your usual discharge. The test can be performed also in cases of itching or irritation, with a normal discharge. test sensitivity 91.8% and test specificity 92.9%.

### SUMMARY AND EXPLANATION OF THE TEST

Vaginal infections are a widespread problem among women of all age groups. while some vaginal infection cases are accompanied with complaints of an abnormal vaginal discharge, close to 50% of infection cases have no such symptoms. The common causes of Vaginitis are Bacteria, Fungus, and Parasites (Vaginitis may be caused by a combination of such infections). Bacterial Vaginosis may cause complications, especially in the course of pregnancy and childbirth (1,2,3).

The VS-SENSE indicates by color change (on the yellow tip of the VS-SENSE swab) whether your vaginal secretion has disordered acidity parameters. Results are based on a combined indication of pH level and buffer capacity (watery secretion), which characterizes bacterial and/or parasitic infection (4,5,6).

A positive test indicates an elevated vaginal pH level, know to accompany bacterial and parasitic infections, as well as non-infectious causes (including estrogen deficiency).

### INDICATION FOR USE

The VS-SENSE should be used to indicate whether your abnormal vaginal discharge is caused by bacterial or parasitic infections. The VS-SENSE test determines whether there is a high or low risk of bacterial or parasitic infections by indicating disordered acidity parameters.

An indication of abnormal acidity of the vaginal discharge can help evaluate whether your vaginal symptoms are likely caused by an infection that may require follow-up with your healthcare provider. Some vaginal infections are known to be associated with abnormal vaginal acidity, have no over-the-counter treatments, and must be treated by your healthcare provider.

### PACKAGE CONTENTS

2 individually foil-wrapped VS-SENSE units.  
1 package insert.  
keep the 2 individually foil-wrapped VS-SENSE units sealed and use it immediately after opening. Time watch is not supplied.  
The user may use her own watch.

### CONDITIONS & LIMITATIONS FOR USE

- To ensure valid results, do not perform the VS-SENSE if the answer to any of the following questions is yes: (a) is it less than one day before or the day after your period; (b) there are signs of menstruations or any vaginal bleeding; (c) it is less than 12 hours after sexual intercourse or vaginal douching.
- Do not insert the swab deeper than the handling point.
- If you are pregnant, the VS-SENSE may indicate an amniotic fluid leakage. If you are pregnant, consult your physician before use, as interpreting test results during pregnancy requires professional knowledge.
- Some menopausal women may have an elevated vaginal pH, and may there for receive a green/blue result with the swab even though they are not suffering from bacterial vaginosis or trichomoniasis. As such, you should not use the test if you are menopausal.

### DIRECTIONS FOR USE/TEST PROCEDURE

- Take one VS-SENSE out of the bulk package.
- Unwrap the individual VS-SENSE (swab). If the swab is damaged or the swab tip is stained, dispose the test and use a new one.
- Make sure that the swab tip dose not contact anything before insertion into the vagina entrance.
- Hold the VS-SENSE test with your fingers on the handling point (pressed section) of the swab.

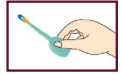
- Separate the vaginal outer layer skin so the vagina is exposed.

- Insert the yellow tip of the VS-SENSE until your fingers reach the outer skin of the vagina (approximately 5cm (2") into the mid vagina) and rotate the swab several times. Do not touch the vaginal entrance.
- Withdraw the VS-SENSE from your vagina. Carefully avoid the tip from contact with anything, and check that it has collected a visible amount of secretion.
- Wait 10 seconds. Check the tip of the VS-SENSE for color changes from yellow to blue or green. Note: if any blood stain appears on the VS-SENSE, disregard the results.
- Dispose the VS-SENSE after usage as you would any other feminine hygiene product.

### INTERPRETATION OF RESULTS

Positive Result:

If the tip is stained blue or green after 10 seconds the risk of having a bacterial or parasitic infection is high. Any Partial blue or green stain on the yellow indicator swab tip should be considered as a positive result.

**Important: If you are pregnant and the tip is stained blue or green, this may indicate an amniotic fluid leak.**

Negative Result

If the tip is not stained after 10 seconds, the risk of having a bacterial or parasitic infection is low.

### REPEATING THE TEST

If you are not certain that a vaginal discharge was collected on the tip, perform a second sampling. If the tip stained blue or green on either test, the risk of having a bacterial or parasitic infection is high

### REPORTING RESULTS TO YOUR PHYSICIAN

Positive results should be reported to the physician.

### STORAGE AND STABILITY

Store in a dry place and at room temperature. Please note the expiration date on the individual test package.

DYN DIAGNOSTICS LTD. 7 Ha'eshel St. P.O.B 3063 Caesarea Industriila park 3088900  
Customer Service: 1-800-333-636

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### REFERENCES:

- Owen MK, Clenney TL. Management of vaginitis. Am Fam Physician. 2004 Dec 1;70(11):2125-32.
- Leitch H, Bodner-Adler B, Brunbauer M, Kaider A, Egarter C, Husslein P. Bacterial vaginosis as a risk factor for preterm delivery: a meta-analysis. Am J Obstet Gynecol. 2003 Jul;189(1):139-47.
- Yen S, Shafer MA, Moncada J, Campbell CL, Flinn SD, Boyer CB. Bacterial vaginosis in sexually experienced and non-sexually experienced young women entering the military. Obstet Gynecol. 2003 Nov;102(5 Pt 1):927-33.
- Pavletic AJ, Hawes SE, Geske JA, Bringe K, Polack SH. Experience with routine vaginal pH testing in a family practice setting. Infect Dis Obstet Gynecol. 2004 Jun;12(2):63-8.
- Thinkhamrop J, Lumbiganon P, Thongkraraj P, Chongsomchai C, Pakarasang M. Vaginal fluid pH as a screening test for vaginitis. Int J Gynaecol Obstet. 1999 Aug;66(2):143-8.
- Egan ME, Lipsky MS. Diagnosis of vaginitis. Am Fam Physician. 2000 Sep 1;62(5):1095-104.
- CDC - Sexually Transmitted Diseases Treatment Guidelines, 2010 Recommendations and Reports December 17, 2010/Vol. 59/No. RR-12

Manufactured: Common Sense Ltd. Haeshel St. 7, P.O.B. 3567, Caesarea 3088900, Israel.  
www.cs-commonsense.com  
EU Authorized Representative : Obelis s.a, Boulevard Général Wahis 53, 1030. Brussels,  
Belgium, www.obelis.net  
FPCSVDY0002

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