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## Manufacturer

**Shenzhen Bi-rich Medical Devices Co.,Ltd**

**Address: No. 10th, Hongqian 2nd Road ,ShangNan, Shajing street, Bao'An district, 518125 Shenzhen City, Guangdong Province,P.R.China**

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## IEC Symbols

---

Caution Attention or Consult accompanying documents.....	
Type BF Equipment.....	
Alternating Current.....	
Double Insulation.....	
Indoor use only.....	

## Important Safeguards

---

### **Caution:**

- 1.Federal Law restrict this device for sale or use on the order of a physician.
- 2.Follow the instruction of your physician to operate this unit.
- 3.This product is a nebulizer for the inhalation of medical aerosols and is suitable for children used solutions. Use only the type and amount of medication prescribed by patient's doctor.
- 4.This product is intended for aerosol therapy only. And other use is not recommended.
- 5.Do not untwist the jet nebulizer during operation.
- 6.Do not pour more than 6ml of solution into the jet nebulizer.

### **Danger: To Reduce the risk of electrocution.**

- 7.Always unplug the unit immediately after using.
- 8.Do not use while bathing.
- 9.Do not place or store the unit where it can fall or be pulled into a tub or sink.
- 10.Do not place or drop into water or other liquid.
- 11.Do not reach for a product that has fallen into water .Unplug immediately.

### **Warning :To Reduce the risk of burns, electrocution, fire or injury**

1. Electrical Shock Hazard—do not remove the cabinet or open the cover.
2. Disconnect the power cord from the electrical outlet before cleaning or servicing.
3. Do not place this equipment near hot, sparking or burning objects.
4. Do not use oil or grease on or near this device.
5. Turn the unit off when it is not in use.
6. Keep the cord away from HEATED or HOT surfaces.
7. NEVER drop or insert any object into any opening.
8. NEVER block the air openings of the product or place it on a soft surface ,such as a bed or couch, where the air openings may be blocked.
9. Avoid operating in wet or damp locations.
10. Unplug the unit before filling the nebulizer.
11. When using this product near TV, microwave oven ,pulse telephone X ray or other strong electric field, it will be disturbed, suggested to be far away from these device measuring.

**⚠ Warning: To reduce the risk of infection:**

1. This Nebulizer kit is intended for both adult and pediatric patient use.
2. Cleaning of the nebulizer is recommended after each aerosol treatment. disinfecting is recommended once a day. Please follow cleaning and disinfecting instructions in this manual.

This unit is not suitable for use in anaesthetic breathing systems

This unit is not suitable in suspension or high viscosity form. In such cases information should be sought from the drug supplier

## Introduction

This compressor nebulizer is designed to deliver the prescribed medication solution to treat patient respiratory disorders, such as asthma, allergies and bronchitis. The nebulizer converts the medication solution into an aerosolized mist which is inhaled by the patient through the mouthpiece or masks. Read this manual thoroughly before using the nebulizer and save for future reference

## Specification

Electrical Requirements	220V AC/50Hz
Power Consumption	Below 60W
Rated Current	0.7 A
Dimensions	Length 150mm × Width 140mm × Height 90mm
Weight	1.2kg(2.64lbs)
Sound Pressure Level	55 dB(A)
Max.Comp.Pressure	35 Psi to 50psi (241 kpa to 345 kpa)
Neb.Operating.Pressure	9~16Psi (62kpa to 110kpa)
Liter Flow Range	5-8L/Min
Average Nebulization Rate:	around 0.33 ml / Min( 6ml medicines can be nebulized within 18.0 minutes)
Maximum nebulizer solution capacity	6ml
Maximum residual medicine volume	0.5 ml
Operating Temperature	+ 10℃ to + 40℃
Operating Humidity	30% RH to 85% RH
Storage/Transport Temperature Range	-20℃ to + 70℃
Storage/Transport Humidity	10% RH to 95% RH
Operating and Storage/Transport Atmospheric Pressure	700mb to 1,060mb

## Protection against electric shock

☐ -classified as Type II

☒ -Type BF applied parts: Mouthpiece, masks

Degree of safety in the presence of flammable anesthetics or oxygen


—No AP/APG (not suitable for use in the presence of flammable anesthetics or oxygen)Mode of operation—Continuous

IPX0—Protection against harmful in gress of water is Ordinary

## Description

---

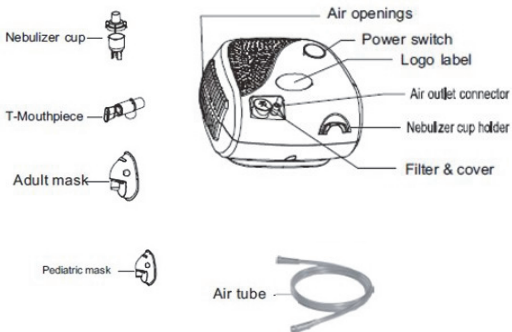1. Air outlet connector
2. Filter cap
3. Power switch
4. Power cord
5. T-mouthpice
6. Nebulizer cup
7. Air inlet connector
8. Tubing
9. Foam filters

## Operation Instructions

---

1. Place compressor on a stable, sturdy and flat surface so that the unit can be easily reached when you are seated.
2. Pull out the power cord from the base of the unit ,and make sure that the unit is in the " off" (O) position by pressing on the left side of switch.
3. Plug power cord into a wall outlet.
4. Connect one end of the nebulizer tubing into the air outlet connector.
5. Add the prescribed solution through the opening on cap using an eye dropper or pre-measured dose container.  
And the marking "6cc" marked on the nebulizer cup is the maximum level  
And the medication can not be filled beyond the maximum level
6. Assemble mouthpiece and T-mouthpiece, and insert into the top of the nebulizer cap. If an aerosol mask is used ,connect the bottom of mask to the top of nebulizer cap.
7. Connect tubing to nebulizer air inlet connector.
8. Turn the power on to start the unit.
9. Implement the treatment by placing the mouthpiece between the teeth. Inhale and exhale through the mouthpiece.
10. If an aerosol mask is used, place the mask over the mouth and nose.

Cleaning

## Cleaning

---

1. Turn the power off and unplug from the wall outlet .
2. Remove the tubing from the air inlet.
3. TO CLEAN: Disassemble mouthpiece, T-mouthpiece, nebulizer cup, chamber and baffle, and wash this items in hot water with dish-washing detergent  
Rinse these items thoroughly to remove the detergent and let air dry.
4. TO DISINFECT: Mix one part of white vinegar with 3 parts hot water in a clean container. Submerge mouthpiece, T-mouthpiece, nebulizer cap, chamber and baffle for half an hour in the solution. Remove from solution and let air dry.
5. There is no need to clean the tubing. If necessary wipe the surface regularly.

**CAUTION:**The jet nebulizer must be changed if it gets clogged.

**CAUTION:**The nebulizer and accessories must not be boiled.

## Maintenance

---

1. The filter of air compressor should be replaced when it turns gray.

Additional filters can be procured from your dealer.

2. Only authorized personnel shall make repairs to this product.

**CAUTION:** The nebulizer may be damaged if used with a dirty filter or if the filter is replaced by other material such as cotton. Do NOT operate without a filter.

## Warranty

---

Shenzhen Bi-rich Medical Devices Co., Ltd warrants the nebulizer compressor to be free from defective workmanship and materials for a period of (3)year from the date of purchase. Check with your dealer for more details. This warranty is limited to the original purchaser of the compressor. Any defective part or assembly will be repaired or replaced at the sole discretion and determination of Shenzhen Bi-rich Medical Devices Co., Ltd if the unit has been properly operated during the warranty period. Normal maintenance items and disposable components are not covered by this warranty. Shipping charges if any, shall be paid by the purchaser.

Note: There is no other express warranty. Implied warranties including those of merchantability and fitness for a particular purpose are limited to the duration of the express limited warranty and to the extent permitted by law and all implied warranties are excluded. This warranty does not cover providing a loaner compressor, compensating for costs incurred for compressor rental or labor costs incurred in repairing or replacing defective part(s).

## Technical data

Particle Size: \*\*MMAD \* approximately 3  $\mu\text{m}$

Appropriate Medication Quantities: 2ml minimum -6 maximum

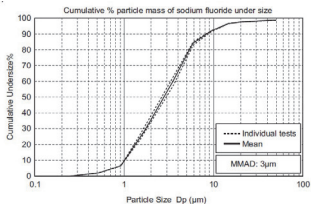
Nebulization rate: 0.4ml/min (by weight loss)

Aerosol output - 0.4 ml / Minute

Aerosol output rate - 0.066 ml / Second

Bi-rich Compressor nebulizer the particle size distribution curve as well:

Result of cascade impactor \*\*measurements for particle size with Bi-rich Compressor nebulizer and nebulizer kits

MMAD = Mass Median Aerodynamic Diameter

## Disposal

---

Once the product life has ended or its components do not work anymore  
Their disposal should be carried out according to the current regulations

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You can find an Instructional video in our  
site- <http://www.dyn.co.il>

## Manufacturer

**Shenzhen Bi-rich Medical Devices Co.,Ltd**  
Address: No. 10th, Hongqian 2nd Road ,ShangNan, Shajing  
street, Bao'An district, 518125 Shenzhen City, Guangdong  
Province, P.R.China

Doc. No.: BR/3-QW-DR-28  
Version No.: A/1