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
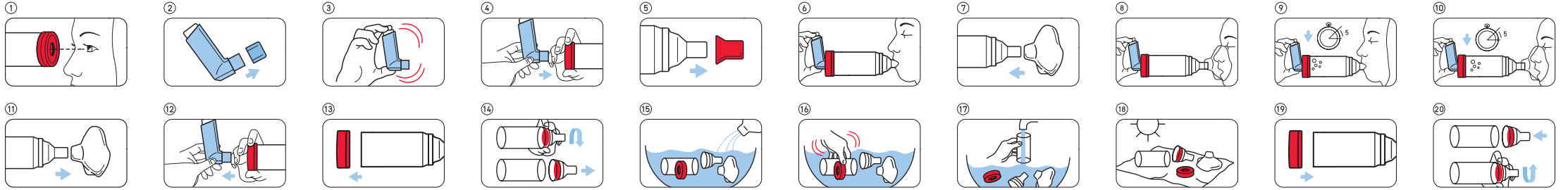
PRESENTATION

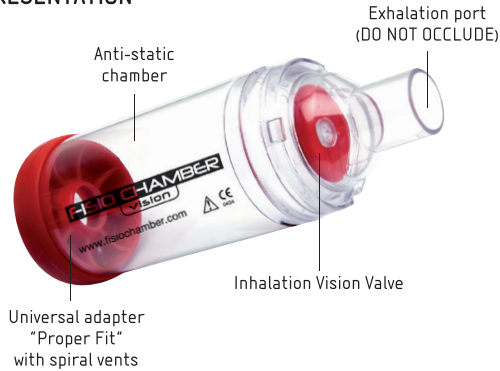PRECAUTIONS

1. Before use, carefully check all the parts, valves and holes ①. Replace if any defect is noticed.
2. Remove cap from MDI (Metered Dose Inhaler) ②. Shake the MDI per instructions ③. Insert and seal the MDI into the universal adapter "Proper Fit" ④.
3. Remove cap from Chamber. Apply the mouthpiece into the mouth and verify to have properly sealed. OR According to the patient's needs, you can insert the Fisio Mask into mouthpiece of the chamber. Verify the mask safety valve is openable (if not, replace it). Apply the mask to the face. Verify to have properly sealed. Inhale and exhale normally.

Then press the MDI one time and start to inhale slowly. Carefully look at the VISION VALVE as the movement shows the inhalation act. Maintain seal the mouthpiece (or the mask, if used) and breath for at least 5 inhalation acts after MDI is pressed ⑤, ⑥ or ⑦, ⑧. After every inhale, hold your breath for at least 5 seconds to allow the medicine deposit in the lungs. Wait before a new puff and repeat this step as for doctor or MDI instructions. Always follow prescribed dosage ⑨, ⑩.

CLEANING INSTRUCTIONS

Clean before first use, then at least once a week following instructions.

1. Remove parts as shown and open the chamber ⑪ - ⑭.
2. Soak the parts for 15 minutes in lukewarm clean water and mild detergent ⑮. Agitate gently ⑯.
3. Rinse the parts in clean water ⑰. Shake out excess of water and allow to air dry ⑱. Do not use dry clothes to dry. Make sure all the parts are dry before reassembly. Reassembly parts as shown and stock the product in a dry and clean container ⑲, ⑳. Storage Temperature $-10^{\circ}\text{C} / +45^{\circ}\text{C}$ Relative Humidity 30% / 80%.

CAUTIONS

Follow instructions before every use. Do not leave Fisio Chamber Vision or part of it unattended with children, as some components are small enough to be swallowed. This product is a Single Patient Use. Do not use with different patients to prevent contaminations. The product

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should be replaced after max. 2 years of continual use. Replace the product if damage, sealing of the valve, environmental condition, storage and frequency of use make the product different from the original use. Dispose according regulations.

The use and the sale of Fisio Chamber Vision is restricted by or on the order of physician. Monitor the patient during use.

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